**FCI PROTECT®**  
**REF : 2138 315 15 B**

FR .....	2
EN .....	3
DE .....	4
IT .....	5
ES .....	6
PT .....	7
NL .....	8
DA .....	9
FI .....	10
SE .....	11
TR .....	12
ZH .....	13
EL .....	14
PL .....	15
SL .....	16
SK .....	17
<b>Figures</b> .....	18
<b>Symbols</b> .....	19

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## FCI PROTECT®

### Description:

FCI Protect® is an isotonic, clear, viscoelastic and sterile solution for intraocular application. As an auxiliary device during surgery of the anterior segment of the eye, FCI Protect® maintains the depth of the anterior chamber and protects the surrounding intraocular tissue.

FCI Protect® contains 2.0% (S5.9100) or 2.4% (S5.9110) hydroxypropylmethylcellulosum and supports the intraocular manipulations due to its viscoelastic and moistening properties. Due to its solubility to water, FCI Protect® can be removed at the end of surgery simply by irrigation and aspiration.

### Composition: 1 ml contains

	S5.9100 FCI Protect® 2.0%	S5.9110 FCI Protect® 2.4%
Hydroxypropylmethylcellulosum	20.00 mg	24.00 mg
Sodium chloride	6.40 mg	6.40 mg
Potassium chloride	0.75 mg	0.75 mg
Calcium chloride 2H <sub>2</sub> O	0.48 mg	0.48 mg
Magnesium chloride 6H <sub>2</sub> O	0.30 mg	0.30 mg
Sodium acetate 3H <sub>2</sub> O	3.90 mg	3.90 mg
Sodium citrate 2H <sub>2</sub> O	1.70 mg	1.70 mg
Other components:	Sufficient quantity of HCl or NaOH, water for injection	

### Packaging:

FCI Protect® contains in their final pouch, packaged in a cardboard box:

- a sterile single-use syringe with 2ml solution ;
- a sterile single-use Luer-Lock injection cannula.

The two elements must be assembled before use.

### Indications:

FCI Protect® serves as a volume substitute for aqueous humor during intraocular operations, such as lens extraction and insertion of an intraocular lens. FCI Protect® maintains the depth of the anterior chamber during surgery and reduces the danger of traumatization of the corneal endothelium, the iris and the ciliary body by direct contact with surgical instruments.

### Mode of action:

FCI Protect® is carefully injected into the anterior chamber using the supplied sterile single-use cannula or a similar cannula (20-25 gauge/Luer-Lock). It is recommended for intraocular lens implantation to cover the implant and instruments with FCI Protect® immediately before surgery. This measure is to additionally protect the endothelium and the surrounding tissue. The FCI Protect® volume to be injected varies and depends on the type of surgery. As a substitute for viscoelastic lost by flow or irrigation, FCI Protect® can be injected several times. At the end of surgery, FCI Protect® must be removed completely using a suitable irrigation/aspiration device. Any residue that has not been removed from the anterior chamber will be flushed out naturally via the trabecular system and Schlemm's canal.

### Warning and contraindications:

FCI Protect® should not be used in cases of a hyper-sensitivity against hydroxypropylmethylcellulosum, as it might endanger the patient's health. Unexpected side effects and complications related to FCI Protect® must be reported to FCI.

### Instructions for use:

Hold the Luer-Lock adapter as shown in (1). Twist the cap carefully with the other hand in an anti-clockwise direction (2). Then remove the cap as shown in (3). Do not use a syringe if the cap is not on properly in its sterile packaging.

### Precautions for use :

FCI Protect® is designed for use by ophthalmologists only.

All normal precautions for intraocular surgery must be observed. FCI Protect® and the supplied sterile single-use cannula are designed for single intraocular application only.

Prior to the application of FCI Protect®, the integrity of the primary package must be controlled to guarantee the sterility and thus the patient's safety.

After surgery with FCI Protect®, a transient post-operative intraocular pressure increase may occur. The intraocular pressure can be decreased again using a suitable therapy.

### Shelflife and stability:

Do not use FCI Protect® after the expiration date as it might compromise the product's efficiency and the patient's safety.

### Storage instructions:

Store FCI Protect® at room temperature protected from light.

### For use during surgery:

Prior to removing the cannula from the eye, do not exert any more pressure onto the plunger, in order to avoid the aspiration of an air bubble into the cannula.

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